


# Viewing Cases Using a Web Browser Quick Start Guide

All releases    ■ Viewing cases and listening to recorded audio conclusions

## Contents

Using a web browser to view cases.....	1
Getting Help and Information.....	3

## Using a web browser to view cases

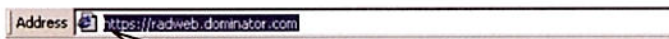

- 1 Get the web site address from your hospital or facility.  
Type it in the address line of your Internet browser (such as Internet Explorer).

Note: The address always starts with **https:**



- 2 Log on by entering your user name and password.  
  
*Note: You may need to log on again before accessing patient records.*

### Welcome to ABC Imaging

The electronic patient records in this website originate from ABC Imaging. These patient records are only for the use of authorized physicians, nurses, employees, students, or staff who have signed confidentiality agreements on file with the entity from which these records are obtained. These records contain information which is privileged, confidential and exempt from disclosure under applicable law. If the reader of this message is not the intended recipient, you are hereby notified that any dissemination, distribution or copying of this communication is strictly prohibited. Browsing for information regarding patients that are not under your immediate care is strictly prohibited, as is tampering or otherwise interfering with the electronic information delivery system.

Patient care decisions should not be based solely on the medical images or audio report summaries that can be accessed here. Integration of clinical and laboratory information, review of the final radiology report, and, when appropriate, consultation with the radiologist(s) are recommended.

For assistance, call: (999) 999-9999

[Click here to view exams](#)

To view exams using Internet Explorer. Does not require a software download.

---


#### Web Ambassador

For clinical review of exam images, reports and audio recording.

- **Download:** Click [here](#) to download Web Ambassador for initial installation or upgrade.


- 3 At the Welcome screen, click the hyperlink: "[Click here to view exams.](#)"

Click the hyperlink.

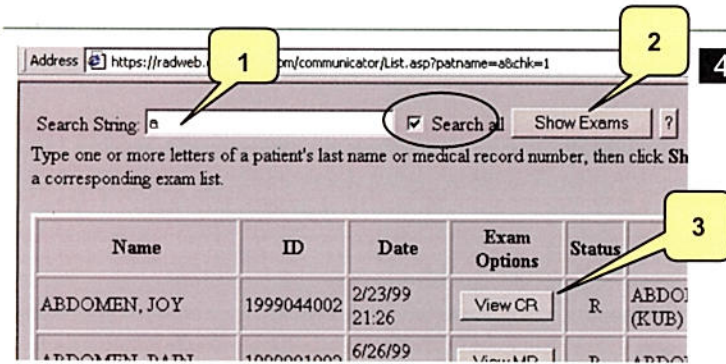


- 4 To select and view an exam:
1. Type the patient's last name or the medical record number.
  2. Click **Show Exams**.
  3. Click **View** in the Exam Options column.

**Note:** The Search all checkbox is only available if you have access to search for all patients.

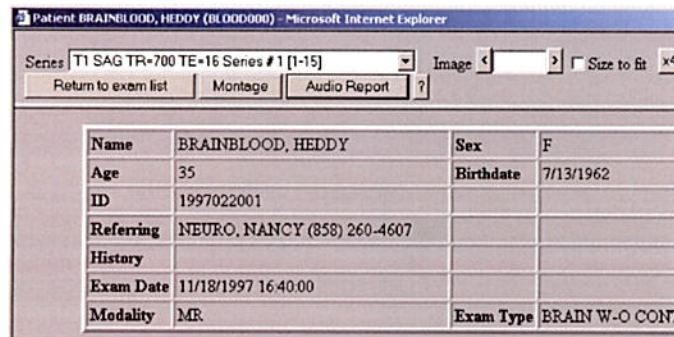

- 5 In the electronic requisition, select:

- Series
- Montage
- Audio report

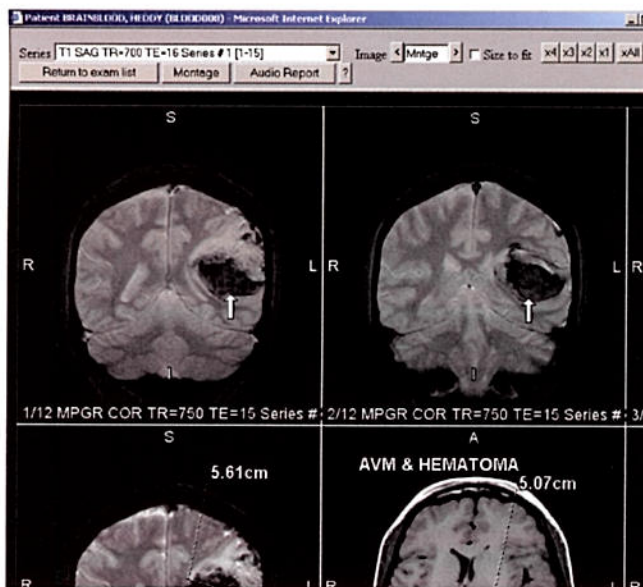

- 6 Click **Montage** to view a set of images that the radiologist has selected and annotated.

In the Exam List or electronic requisition, click ? for Help.

- 7 Display **Online Help** for details on viewing exams and images.

---

## Getting Help and Information

<b>Displaying Online Help</b> Display a comprehensive Help program: <ul style="list-style-type: none"><li>▪ Press <b>F1</b> or click <b>Help Topics</b> from the <b>Help</b> menu.</li></ul>	<b>Accessing the Resource Center</b> View and download white papers, user guides, TechNotes, technical specifications and other documents: <ul style="list-style-type: none"><li>▪ Go to <a href="http://www.dominator.com">www.dominator.com</a></li><li>▪ Click the <b>Resource Center</b> button.</li></ul>
<b>Contacting DR Systems Customer Support</b> <ul style="list-style-type: none"><li>▪ Phone: 1-800-794-5955</li><li>▪ Fax: 1-858-625-3337</li><li>▪ E-mail: <a href="mailto:drsupport@dominator.com">drsupport@dominator.com</a></li><li>▪ Online: <a href="http://www.dominator.com">www.dominator.com</a> Click <b>Customer Support Login</b></li></ul>	